


Typenex Medical

flexiBLOOD Recipient Verification System
Training Guide

Condensed Form + Band

blood band assembly


1. **VERIFY** the patient identity

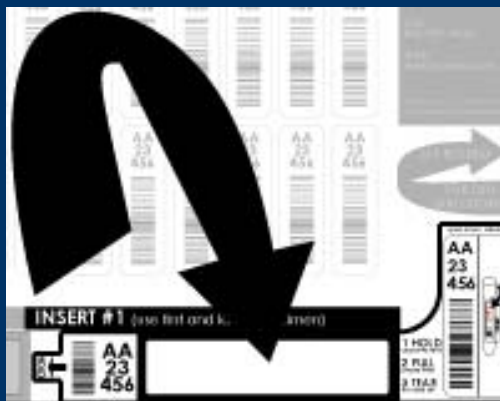

2. **PLACE** facility required patient information onto the Insert within the colored outline: Use a pre-printed information label or handwrite with ballpoint pen

3. **SEPARATE** the Insert from the form at the perforation

blood band assembly

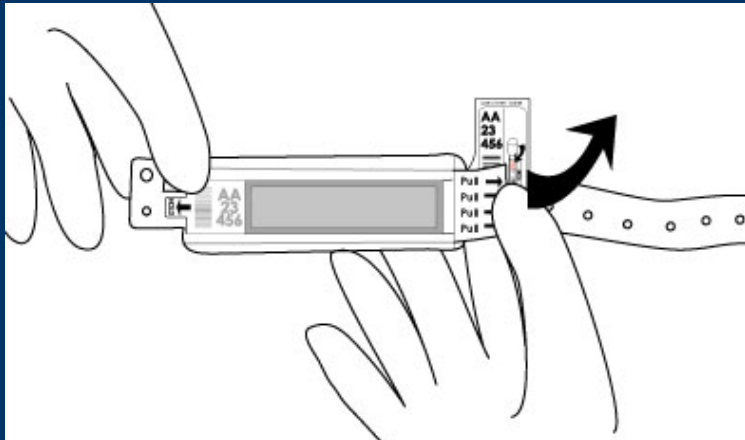


4. **INSERT** tab end of the Insert into opening of pocket (patient information up), behind paper liner that says, "PULL," until tab comes out other end
5. **HOLD** the band and tab
6. **PULL** paper liner out of the pocket, where it says, "PULL"
7. **PRESS** adhesive within band down to seal the Insert into the pocket

blood band application

1. **WRAP** tail (with holes) around patient extremity to size, with two-finger spacing
 2. **SNAP** button closure to secure, cut excess material with scissors
-

specimen labeling

1. LABEL patient specimen tube

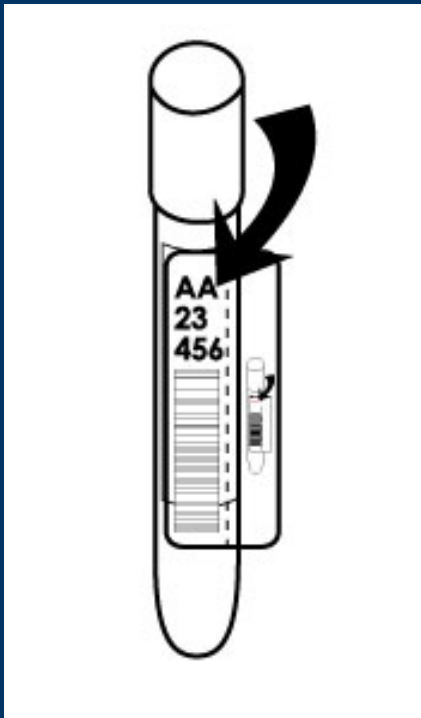

- Peel the BBID sticker from the Insert, on patient band
- Apply lengthwise to tube
- Align a pre-printed label with the dashed line and apply to specimen

specimen labeling

2. **LABEL** additional paperwork, as necessary with the “Extra Labels” and transfer to blood bank
-

recipient verification

1. **VERIFY** positive patient identity and ensure all identifying information matches, then transfuse if everything corresponds correctly
 2. **CONTACT** the blood bank if:
 - The band needs to be replaced, relocated, or resized
 - Any discrepancies between issued unit(s) and patient information exist
-

product information

Product: FLX 002

- flexiBLOOD Condensed Recipient Verification Form
 - Color: Red
 - Latex Free

 - **Domestic Phone:** 866 897 3639
 - **International Phone:** 312 938 3000
 - **Web:** www.typhenex.com
-