

Typenex Medical

flexiBLOOD Recipient Verification System
Training Guide

General Form + Band

blood band assembly


1. **VERIFY** the patient identity



2. **PLACE** facility required patient information onto Insert #1 within the colored outline: Use a pre-printed information label or handwrite with ballpoint pen

3. **SEPARATE** Insert #1 at perforation

blood band assembly

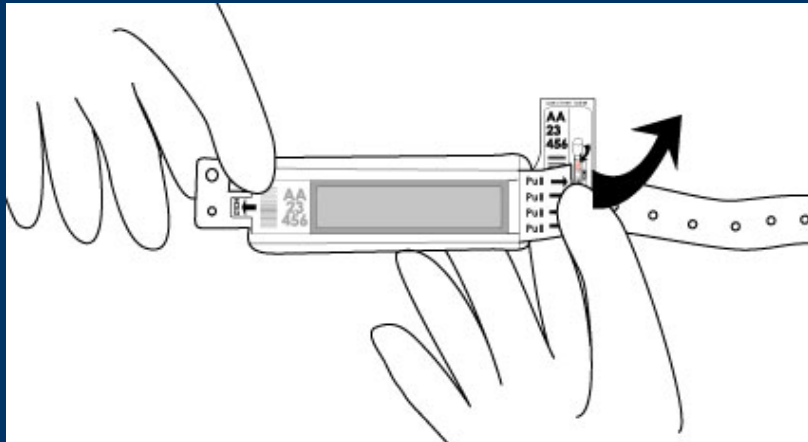

4. **INSERT** tab end of Insert #1 into opening of pocket (patient information up), behind paper liner that says, "PULL," until tab comes out other end
 5. **HOLD** the band and tab
 6. **PULL** paper liner out of the pocket, where it says, "PULL"
 7. **PRESS** adhesive within band down to seal Insert #1 in pocket
-

blood band application


1. **WRAP** tail (with holes) around patient extremity to size, with two-finger spacing
 2. **SNAP** button closure to secure, cut excess material with scissors
-

specimen labeling

1. **LABEL** patient specimen tube, using either of the following methods, depending on your facility protocols:
-

specimen labeling


pre-printed label

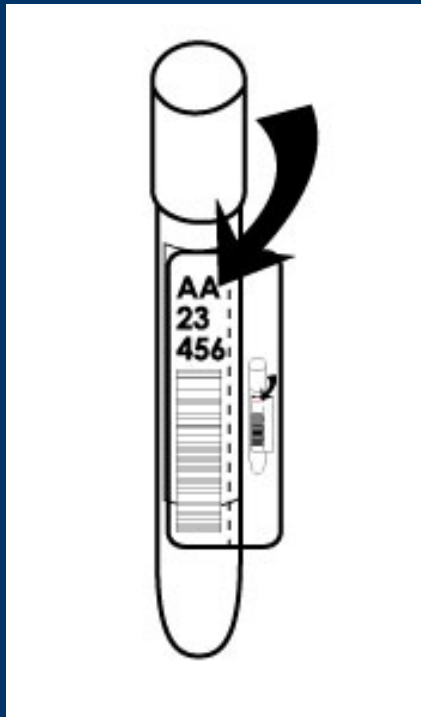

- Peel BBID sticker from Insert #1, on patient
- Apply lengthwise
- Align pre-printed label with dashed line and apply to specimen


specimen labeling

handwriting

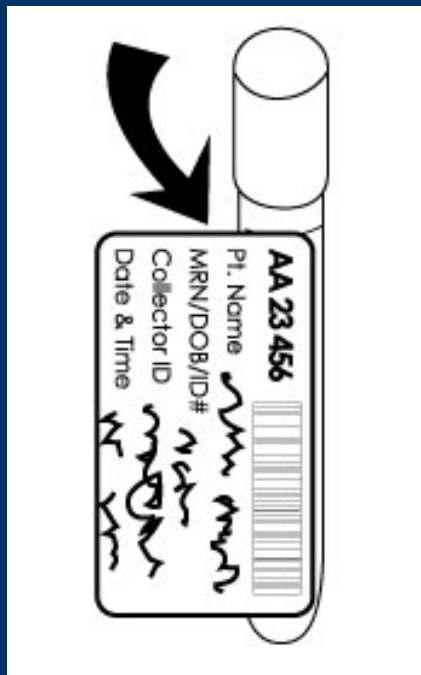

- Complete designated handwriting specimen sticker on the form
- Remove from form to apply lengthwise to specimen

specimen labeling

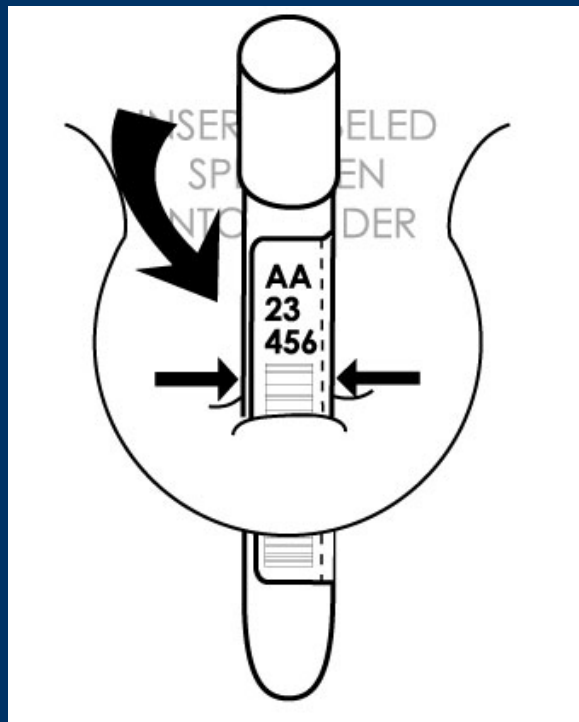

2. LABEL additional paperwork and transfer to blood bank

- "Specimen Holder" can be used to secure a labeled specimen to form

recipient verification

1. **VERIFY** positive patient identity and ensure all identifying information matches, then transfuse if everything corresponds correctly
 2. **CONTACT** the blood bank if:
 - The band needs to be replaced, relocated, or resized
 - Any discrepancies between issued unit(s) and patient information exist
-

product information

Product: FLX 001

- flexiBLOOD General Recipient Verification Form
 - Color: Red
 - Latex Free

 - **Domestic Phone:** 866 897 3639
 - **International Phone:** 312 938 3000
 - **Web:** www.typhenex.com
-