

transfusion process steps:

Patient Identification

Sample Identification

Blood Bank Component

Blood Unit Issue

Recipient Verification

Blood Band Basics

- Accommodates and protects handwritten patient information
- Adhesive closure, no scissors required
- Unique identifier (BBID: Blood Bank ID or armband number) in barcode and alphanumeric format
- Fits most patient sizes
- Compatible with R3 Attachment System to relocate, resize or replace on patient

- 1 specimen tube sticker with unique BBID provides directions for proper overlabeling of the specimen tube with prompted patient information sticker
- 2 attachment adhesive tabs at the end of the detachable stickers affix to specimen for transport to blood bank

- Unique BBID on specimen tube sticker can be scanned into LIS
- 13 total detachable BBID stickers are available for paperwork, transfusion tags, donor unit labels, etc.

- 4 removable BBID stickers left on patient for use with the request for blood
- Label donor units with unique BBID stickers from specimen

- Ensure unique BBID matches on the recipient blood band, donor unit labels and any paperwork at the recipient bedside

- **Highlights:** Specimen tube sticker with directions; Prompted patient information sticker
- **BBID Longevity:** One pre-transfusion specimen
- **Durability:** 72 hours
- **Technology:** GS1-128 barcode
- **Indications:** In-patient use
- **Packaging:** 15 cartons/10 bands, 150 bands/case

specimen tube BBID sticker with directions

detachable BBID stickers (13)

attachment adhesive tabs (3)

removable BBID stickers (4)

adhesive closure shield

permanent sticker

closure slots (3)

prompted patient information sticker

clear tail

protective adhesive shield



band is available in these colors:

- **4R4658 RED**
- **4R4659 GREEN**
- **4R4660 YELLOW**
- **4R4661 BLUE**
- **4R4662 ORANGE**

Enable your unique transfusion process with a Typenex blood band solution. Contact us for more information.

Phone | 866 897 3639
Fax | 866 897 3630
www.typenex.com