

transfusion process steps:

Patient Identification

Sample Identification

Blood Bank Component

Blood Unit Issue

Recipient Verification

Blood Band Basics

- Accommodates and protects pre-printed patient information labels
- Adhesive closure, no scissors required
- Unique identifier (BBID: Blood Bank ID or armband number) in barcode and alphanumeric format
- Fits most patient sizes
- Compatible with R3 Attachment System to relocate, resize or replace on patient

- 1 specimen tube sticker with unique BBID provides directions for proper overlabeling of the specimen tube with pre-printed patient information label
- 2 attachment adhesive tabs at the end of the detachable stickers affix to specimen for transport to blood bank

- Unique BBID on specimen tube sticker can be scanned into LIS
- 13 total detachable BBID stickers are available for paperwork, transfusion tags, donor unit labels, etc.

- 4 removable BBID stickers left on patient for use with the request for blood
- Label donor units with unique BBID stickers from specimen

- Ensure unique BBID matches on the recipient blood band, donor unit labels and any paperwork at the recipient bedside

- **Highlights:** Specimen tube sticker with directions; Protective adhesive shield
- **BBID Longevity:** One pre-transfusion specimen
- **Durability:** 72 hours
- **Technology:** GS1-128 barcode
- **Indications:** In-patient use
- **Packaging:** 15 cartons/10 bands, 150 bands/case

specimen tube BBID sticker with directions

detachable BBID stickers (13)

attachment adhesive tabs (3)

removable BBID stickers (4)

adhesive closure shield

permanent sticker

closure slots (3)


pre-printed patient information label area

clear tail

protective adhesive shield



band is available in these colors:

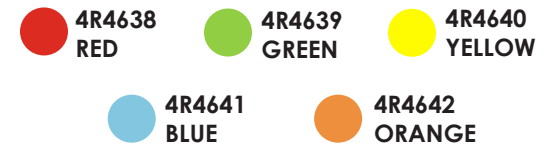

Enable your unique transfusion process with a Typenex blood band solution. Contact us for more information.

Phone | 866 897 3639
Fax | 866 897 3630
www.typenex.com