

Typenex® Original Blood Band

transfusion process steps:

Patient Identification

Sample Identification

Blood Bank Component

Blood Unit Issue

Recipient Verification

Blood Band Basics

- Accommodates handwriting protocols and leaves a copy on patient
- Self-cutting clip closure, no scissors required
- Unique identifier (BBID: Blood Bank ID or armband number) in easy-to-read alphanumeric format
- Fits most patient sizes

- 1 specimen tube sticker with unique BBID accommodates handwritten patient information
- Attachment adhesive tab at the end of the detachable tail of stickers affixes to specimen for transport to blood bank

- Unique BBID on specimen tube sticker can be hand-keyed into LIS
- 14 detachable BBID stickers, and their copies, are available for paperwork, transfusion tags, donor unit labels, etc.

- 2 removable stickers left on patient for use with the request for blood
- Label donor units with unique BBID stickers from specimen

- Ensure unique BBID matches on the recipient blood band, donor unit labels and any paperwork at the recipient bedside

- **Highlights: Self-cutting clip; Easy-to-read alphanumeric BBID**
- **BBID Longevity: One pre-transfusion specimen**
- **Durability: 72 hours**
- **Technology: Human readable, alphanumeric**
- **Indications: In-patient use**
- **Packaging: 15 cartons/10 bands, 150 bands/case**

self-cutting clip closure

removable BBID stickers (2)

specimen tube sticker with prompted patient information area

detachable BBID stickers (14)

attachment adhesive tab (beneath arrows)

band is available in these colors:



Enable your unique transfusion process with a Typenex blood band solution. Contact us for more information.

Phone | 866 897 3639

Fax | 866 897 3630

www.typenex.com