

Typenex Medical


Next Generation Barcode Blood Band
Training Guide

Condensed Version

Step 1: Check Patient ID and Existing BBID Band


- **CONFIRM** patient's identifying information
 - If no existing BBID band on patient:
 - **APPLY** a new BBID band (Step 2)
 - If BBID band with **NO** extra labels left:
 - **CHECK** with blood bank for units on hold for that BBID
 - If BBID band with valid labels available:
 - **OBTAIN** the patient's blood sample (Step 4)
-

Step 2: Date BBID Band

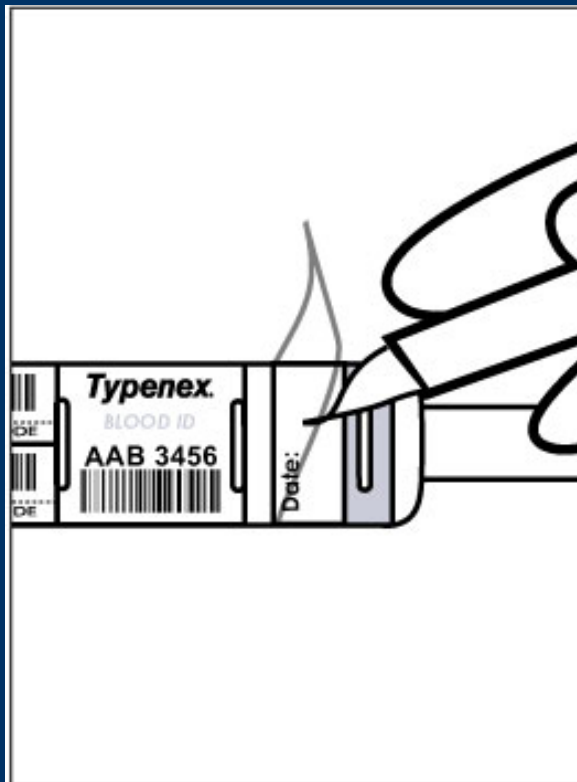


- **WRITE** the date of the specimen collection (and other information, if desired) below the *Date* prompt using a ballpoint pen
- **PROTECT** this information by removing the adhesive shield liner and sticking down

Step 3: Apply BBID Band

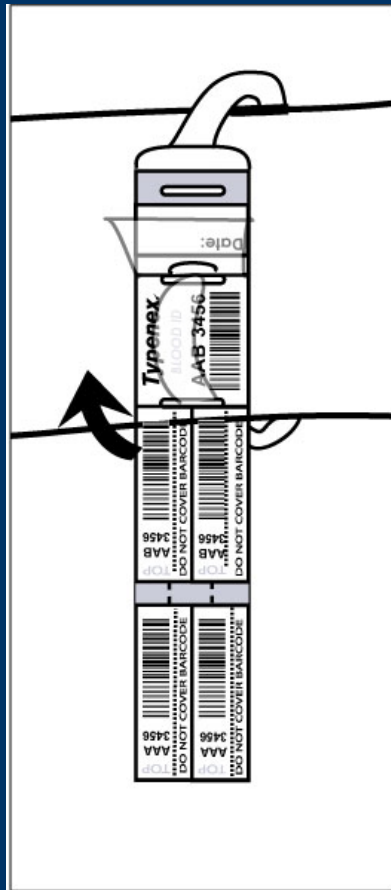


- **PLACE** the date area of the BBID band over the wrist or ankle
- **WRAP** the clear tail around and up through the first closure slot (left of the permanent sticker)
- **LIFT** the adhesive closure shield and **HOLD** it up

Step 3: Apply BBID Band (continued)

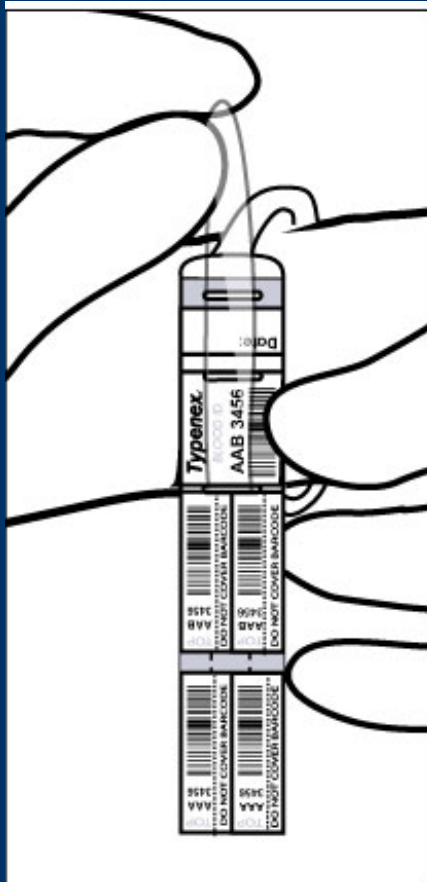


- **PULL** clear tail across the permanent sticker and through the second closure slot in the closure shield
- **PEEL** liner of the closure shield and stick down, covering the clear tail and the permanent sticker
- **CUT** off or **SECURE** excess into slot

Step 4: Label Specimen

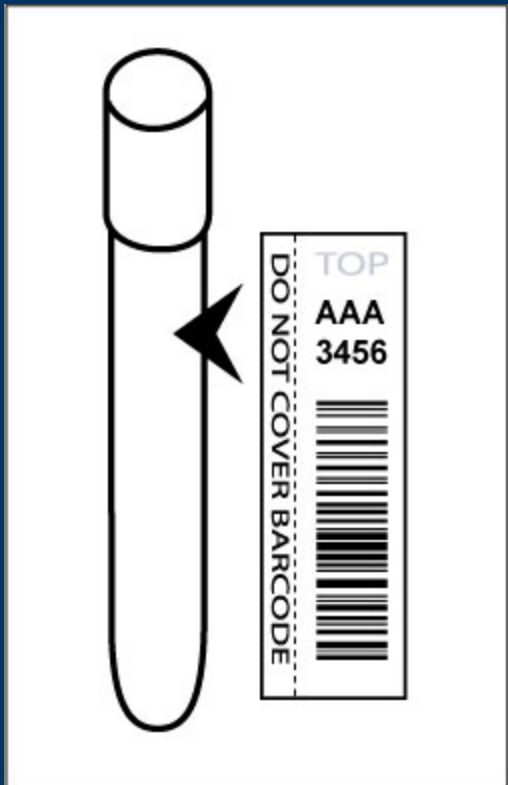


- **PEEL** one barcoded BBID label from the band
- **AFFIX** BBID label, so *TOP* is at the top of the specimen tube

Step 4: Label Specimen (continued)

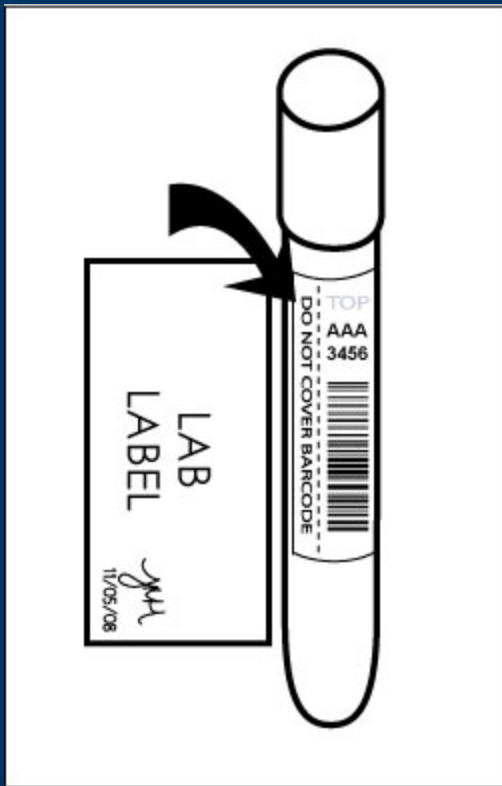


- **PLACE** the designated lab label over the BBID label and align with the dashed line
- **LEAVE** the barcode fully exposed

Step 5: Remove Extra Detachable Sticker & Secure Removable Stickers

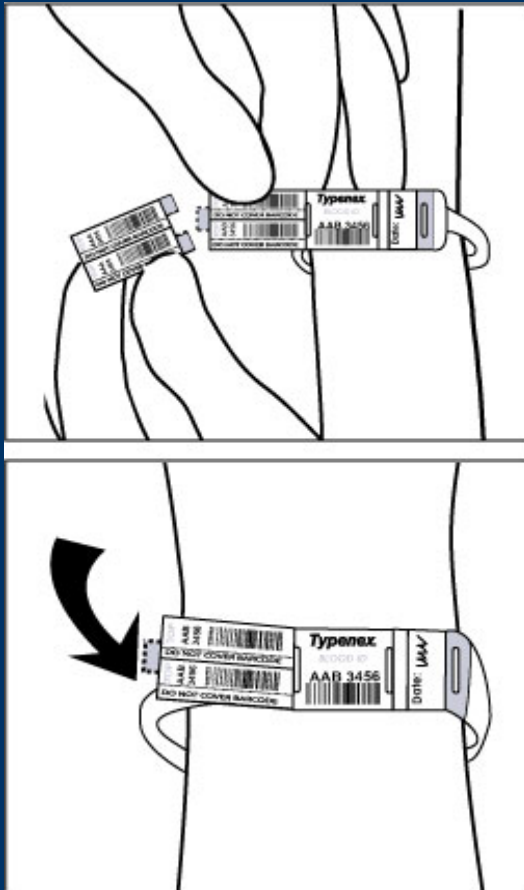

- **REMOVE** the extra detachable sticker from the BBID band
- **USE** adhesive tabs to affix on specimen, if desired; **DISPOSE** of the detached stickers properly if not in use
- **SECURE** the two removable BBID stickers to the clear tail with the red adhesive tab
- If your facility prefers not to use the removable stickers on the patient, **CUT** them off, leaving the closure slot intact

Step 6: Transfuse Patient

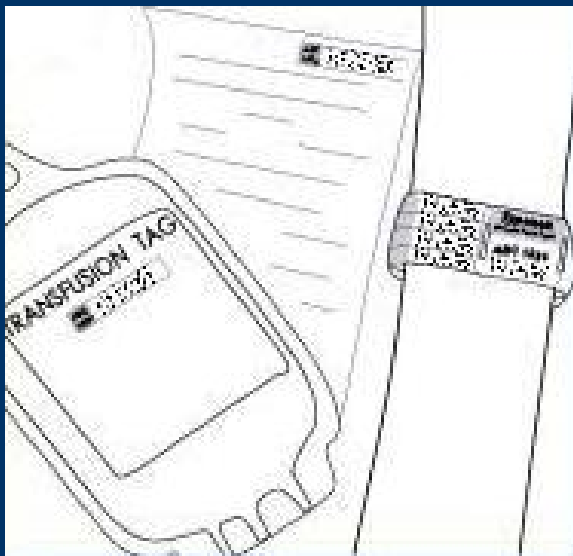

- **CONFIRM** the recipient identity
 - **ENSURE** that the following are matching before transfusing:
 - BBID band unique code
 - BBID unique code on blood bag label/transfusion tag
 - Request for blood
 - **PROCEED** with transfusion
-